



The DUKKI

Pediatric Shower Commode Chair

The daily hygiene care of a special needs child demands the full attention of both parents and caregivers. The Dukki has proven to be an excellent aid for this purpose. It can be used as a commode chair or shower chair for children ages 2 to early teen years. The shape of the seat was developed to offer comfort and relaxation. A comfortable foam inlay can extend the sitting time. The seat can be easily tilted to provide the child with the best position for toileting or showering. Abduction block, armrest and footrest provide proper support. The tray and abduction block are quickly and easily removed for fast transfers.



Features

- Tilt-in-space positioning on an adjustable frame
- Fits over most standard toilets
- Rounded edges for comfort and ease of cleaning
- Optional accessories for positioning and comfort

Components

Dukki comes in 3 sizes, see chart for dimensions. The blue seat is molded plastic with rounded corners for comfort and ease of cleaning. The blue aluminum chassis frame can be adjusted to the height of the toilet. Dukki rolls on dual 3" wheels with swivel casters, front wheels have foot activated locks.

Abduction block snaps onto front of the seat, and channels waste into the toilet or collection bowl; it also serves as a splash guard. Armrest/tray attaches to back without tools and snaps into abduction block. Tray serves as an armrest, and safety support for secure seating. Footrest is height adjustable. A special tool, hidden in the locking pin of the footrest, is used for frame adjustments and to secure the footrest at the chosen height.



Showering



Over the toilet



As a commode chair

CLARKE HEALTH CARE PRODUCTS, INC.

1003 International Drive • Oakdale, PA 15071

Phone (888) 347-4537 • Fax (724) 695-2922 • www.clarkehealthcare.com

The DUKKI



The DUKKI

Pediatric Shower Commode Chair

Accessories

Optional foam inlay, with insert for collection opening, allows for growth and additional cushioning. Curved headrest inserts into the back, is height and angle adjustable without tools. Collection bowl slides into a channel from the back of the chair, holds approx 64 oz. The special chassis will accommodate elongated or raised toilet bowls. A padded belt can be installed in either of 2 positions on the chair back. The belt closes with a plastic buckle.



Specifications

DESCRIPTION	No. 2	No. 3	No. 4
Width of seat (in)	11	12	13
Depth of seat (in)	11	13	15
Height of seat (in)	19-24	18-23	18-23
Width of back (in)	11	11.5	12
Height of back (in)	16.5	18	19
Leg-room, front with narrow abduction block	5	5.5	6
Leg-room, front with wide abduction block	4	5	5
Distance from tray to back support,	8	9	10.5
Tilting adjustment	-5° - 60°	-5° - 60°	-5° - 60°
Height of footrest (in)	9-12	10-15	11.5-17
Max. width of chair (in)	22	23	23.5
Length of chassis (in)	23	26	28
Weight (lb)	22	26.5	29
Capacity (lb)	80	80	80
Product Number	H 21080	H 31080	H 41080

Warranty

Dukki is warranted for 1 year against manufacturer's defects, excluding inlay and upholstered accessories.

CLARKE HEALTH CARE PRODUCTS, INC.

1003 International Drive • Oakdale, PA 15071

Phone (888) 347-4537 • Fax (724) 695-2922 • www.clarkehealthcare.com

The DUKKI