



## IN SHORT

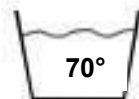

- *Aquatec® Ocean* belt: washing instruction
- *Aquatec® Orca* handling information
- Introduction *Invacare® Hydro* family
- *Aquatec® Orca* upgrade kit

### *Aquatec® Ocean* belt: washing instruction

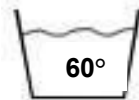

We would like to inform you that we have new washing instructions for our *Aquatec® Ocean* belts.

#### **REGULAR CLEANING:**

Machine wash, hot, 60°C (140°F) using laundry detergent. Tumble dry, or drip dry. Do not iron.

#### **SPECIAL CLEANING:**

For extra sanitizing, the belt can be machine washed, very hot, 70°C (160°F) with bleach.

You can follow these instructions even if the tag shows a max. temperature of 40°.

### *Aquatec® Orca* handling information



**2 sec!**

Please note this *Aquatec® Orca* handling information.

If you would like to change the direction of movement the bathlifter you have to wait for 2 seconds before you press the UP or DOWN button for the opposite direction.

The movement of the bathlifter in the other direction is only possible after the bathlift was stopped for a few seconds.

Please inform your customer about this handling!

## Introduction *Invacare*<sup>®</sup> *Hydro* family



#1521472  
*Invacare*<sup>®</sup> *HydroWave*



#1521473  
*Invacare*<sup>®</sup> *HydroFlex*



#1521474  
*Invacare*<sup>®</sup> *HydroE-Flex*

We are pleased to introduce the *Invacare*<sup>®</sup> *Hydro* family. The chairs are similar to the *Aquatec*<sup>®</sup> *Ocean* family, but the *Invacare*<sup>®</sup> *Hydro* chairs convince with their green and blue color combination which is recommended for people with limited eye-sight.

The technical features and measurement are equal to the referring *Aquatec*<sup>®</sup> *Ocean* modell.

So the *Invacare*<sup>®</sup> *HydroWave* is equivalent to the *Aquatec*<sup>®</sup> *Ocean*, the *Invacare*<sup>®</sup> *HydroFlex* to the *Aquatec*<sup>®</sup> *OceanVIP* and the *Invacare*<sup>®</sup> *HydroE-Flex* corresponds to the *Aquatec*<sup>®</sup> *OceanE-Vip*.

All chairs of the *Invacare*<sup>®</sup> *Hydro* family are including key cut softseat, insert, toilet pan holder and pan.

## *Aquatec*<sup>®</sup> *Orca* upgrade kit



Please note that an upgrade kit for the *Aquatec*<sup>®</sup> *Orca* is now available.

# 1526014  
Upgrade special backrest for *Aquatec*<sup>®</sup> *Orca* with white cover

This item code includes  
#15227 – 2 sideflaps  
#10192 – 4 hinges  
#10715 – 16 blind rivets  
#1471568 – 1 backrest cover white

The assembly instructions are also included.

## CONTACT



Ilse Weis

[iweis@invacare-aquatec.com](mailto:iweis@invacare-aquatec.com)

+49 7562 700-63

Tanja Wiedemann

[twiedemann@invacare-aquatec.com](mailto:twiedemann@invacare-aquatec.com)

+49 7562 700-34