




Mounting at top of cylinder

The cylinder now mounts with the piston facing down. **DO NOT TURN THE PISTON IN THE CYLINDER.** If the piston is turned more than 1/2 turn the piston will open up and the cylinder will not work.




Mounting at bottom of cylinder

Piston will now face downward and cable will attach at bottom. Reattach using existing screws and washers.

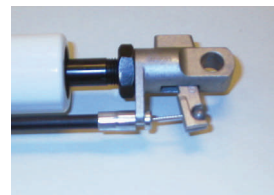

Attaching cable to cylinder bracket

Ball end of cable sits in groove of bracket. The recess of the metal collar on the cable will slip into the bottom of the bracket.

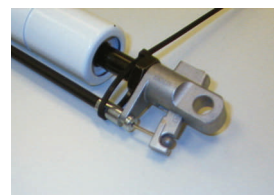

Attaching the cable tie

Use a cable tie to keep the cable in the brackets.



Attaching cable at handle

The ball end of the cable will lay in groove. When the cable is attached on the cylinder the slack will be taken up and the cable will be tight in the handle..