DOLOMITE *ALPHA BASIC*

SV BRUKSANVISNING

GB INSTRUCTIONS

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NO BRUKSANVISNING

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
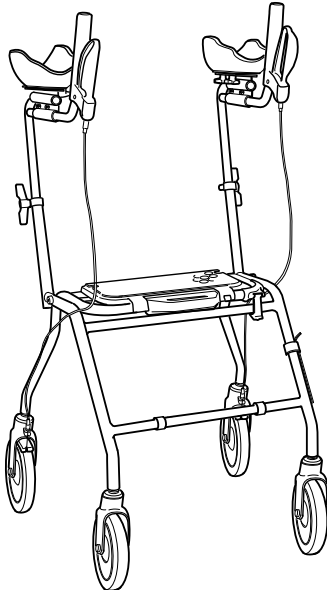
DOLOMITE ALPHA BASIC

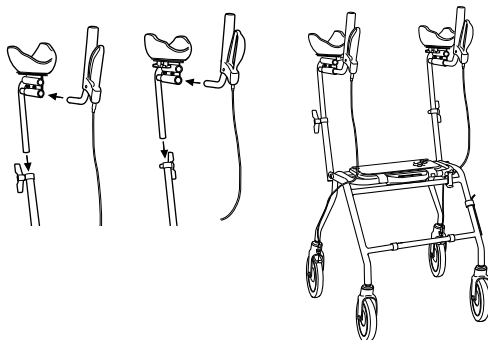SV BRUKSANVISNING

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GB FOLDING/SEAT

- A. Pull the folding blocker and lift the handle for folding. Folded position. Space-saving. Suitable for carrying in stairs. For ready-to-start position, block with thumb for locking. A clicking sound indicates right position. Make sure that the folding blocker is in place when the walker is to be used again.

B. The seat can be folded up to provide a larger walking space, and in order to increase accessibility.

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GB PARKING BRAKE

Press forwards carefully for parking brake. A clear sound is heard when the brake is placed in correct position. N.b. Always place both brakes in parking position. Pull upwards to loosen the brake.

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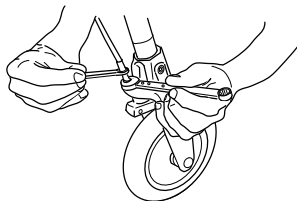
GB BRAKES

Adjust the brake wire using cap keys Nos. 8 and 10 in case of poor brake effect.

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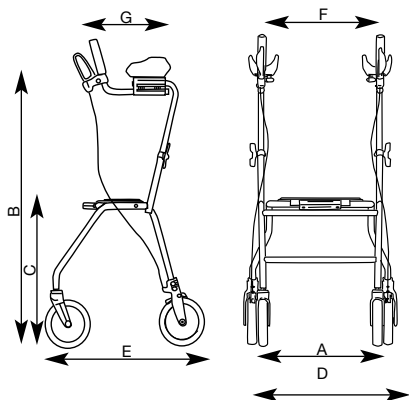
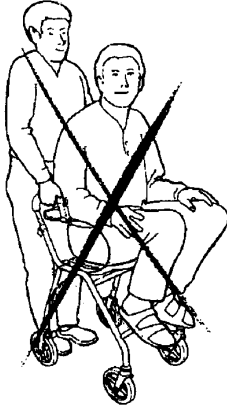
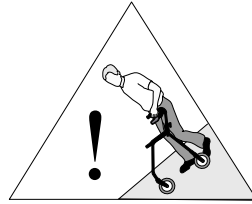
GB A. It is **prohibited** to stand on the walker and to use it for transport of people.
B. Be extra careful when using the walker on a sloping surface.

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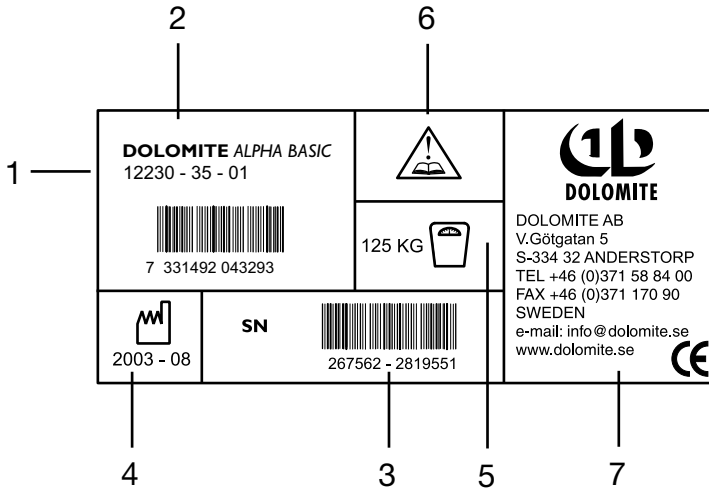
GB

All technical data available on the DOLOMITE ALPHA BASIC label:

- 1: Article number
- 2: Product name
- 3: Serial number
- 4: Date of manufacture
- 5: Maximum user weight
- 6: NB: read the operating instructions
- 7: Manufacturer's contact information.

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GB**INSTRUCTIONS FOR DESTRUCTION OF PRODUCT AND PACKING:**

When the product can no longer be used, it and its accessories as well as the packaging material should be assorted in the correct way. The individual parts can be separated and handled according to the list specification.

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GB MAINTENANCE - SAFETY

To ensure good performance and safety you should regularly check that the following is carried out:

- Clean the rollator with solvent-free household cleaner. Keep the wheels clean.
- Ensure that screws and adjustments are properly tightened.
- Immediately contact the nearest dealer if a fault should appear. Never use a defective walker. Do not attempt repairs yourself!

N.B. Your walker should only be used as a walking aid! User's max. weight: 275 lbs. (125 kg, 20 stones.) **DOLOMITES** will not accept responsibility for adaptations, faulty repairs, etc, that are outside their control.

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DOLOMITE AB
AQUATEC GmbH & Co.KG
MOBITEC Mobilitätshilfen GmbH