# **DOLOMITE**

## *FUTURA, ROBUST, SYMPHONY*

**SV** BRUKSANVISNING

**NO** BRUKSANVISNING

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**GB** OPERATING INSTRUCTIONS

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**FR** MANUEL D' UTILISATION

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FUTURA ISYMPHONY

ROBUST

## FUTURA, ROBUST, SYMPHONY

### SV BRUKSANVISNING

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### GB OPERATING INSTRUCTIONS

Study the operating instructions carefully **before** use. The person who prescribes the product or who is otherwise responsible for the user must ensure that the product is suitable for the user.

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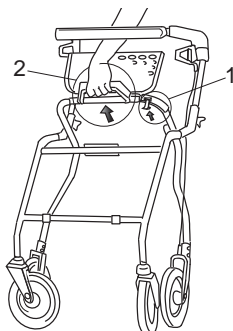
1. Lyft upp lyftspärren.
2. Fäll ihop.

### **GB** FOLDING

1. Lift up the safety latch.
2. Start the handle to fold the walker.  
Folded position.

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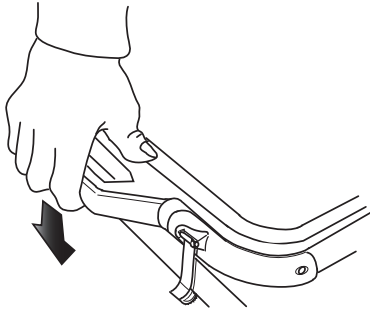
### **GB** OPERATING POSITION

For walking position, lift up handle press down to lock with thumb. A clear sound indicates correct position.

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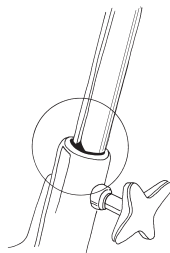
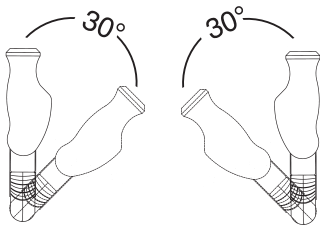
### GB HANDLEBARS

**For adjustment;** Loosen the wing, adjust the handle to suitable height and tighten the wing properly. When standing behind the walker with flat shoulders adjust handles to be at the same height as the user's wrists. Therapist may suggest different heights depending on specific need of the client. If the user unconsciously tends to adjust the height adjustment handles, for safety reason the handles should be tightened with a socket head wrench.

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### GB HANDLES

For optimum performance insure that handles are in the allowed adjustment area at all times. Arrow pointing on the thin inner vertical line indicates maximum allowed outer handle position. By adjustment of the height of handle, do not tighten the handle-locker too tight. The thread can get damaged and then you will not be able to lock the handle.

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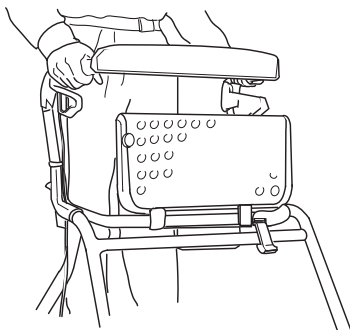
### **GB** FOLDABLE SEAT

The seat is foldable for increased space between the handles.

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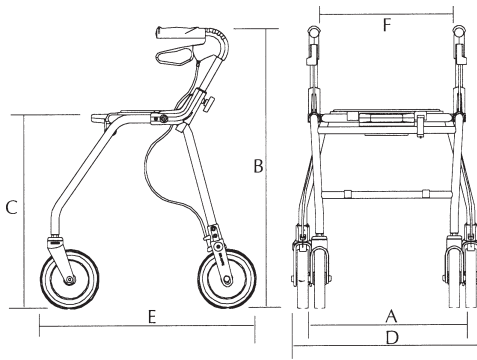
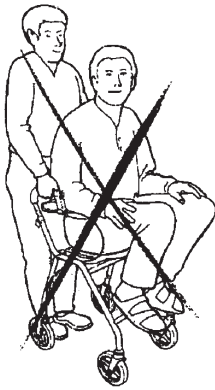
**GB** Not to be used for standing on or as a wheelchair.

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**GB INSTRUCTIONS FOR DESTRUCTION OF PRODUCT AND PACKING:**  
When the product can no longer be used, it and its accessories as well as the packaging material should be assorted in the correct way. The individual parts can be separated and handled according to the list specification.

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**GB MAINTENANCE – SAFETY**

To ensure good performance and safety you should regularly check that the following is carried out:

- Clean the rollator with solvent-free household cleaner. Keep the wheels clean.
- **Ensure that screws and adjustments** are properly tightened.
- **Immediately contact the nearest dealer if a fault should appear. Never use a defective walker.** Do not attempt repairs yourself!

**N.B. Your walker should only be used as a walking aid! User's max. weight: 275 lbs. (125 kg, 20 stones.)** Symphony 100 kg. DOLOMITE will **not** accept responsibility for adaptations, faulty repairs, etc, that are outside their control.

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**DOLomite AB**
**AQUATEC GmbH & Co.KG**
**MOBITEC Mobilitätshilfen GmbH**