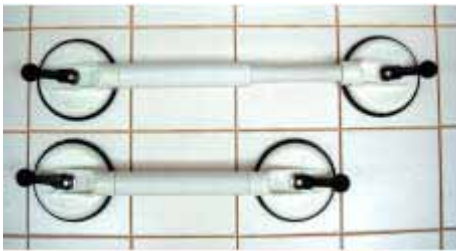
ROTH

Sanitärprogramme®
+ REHA Produkte

CE

Instruction Manual

Mobile Holding Grips

Order No 1403999, 1400521, 1400522, 1400523, 1400526, 1400528

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Attention!

Please read the instructions carefully

- The Mobile Vacuum Suction Grab Rail is intended to assist your body's natural movement, e.g. as an aid for standing up. It is not designed to support your whole body weight. The Mobile Grab Rail is intended to provide support for short periods only.
- Prior to each use, the Mobile Grab Rail should be checked to ensure it is still firmly attached to its underlying surface.
- In so far as is legally admissible, any liability on our part is excluded if the user has not followed the safety rules. Since these safety rules cannot take into account the particular circumstances in each and every individual application, it is the user's responsibility to check before each intended use that the Mobile Grab Rail is suitable for that specific purpose. We do not guarantee features that are not mentioned in our catalogue, leaflet or this instruction manual.
- To avoid any damage or injuries due to the incorrect use of the grab rail and to ensure optimum use from the product, please read the following instructions and hints before use. Please retain the instruction manual and ensure that it is at the disposal of all other users of the rail. If you have additional questions or problems, please contact your supplier.
- We hope you experience trouble-free use and that this product makes daily living easier.

Declaration of Conformity

Roth GmbH, as the manufacturer with sole responsibility, hereby declares that the Mobile Vacuum Suction Grab Rail conforms to the requirements of European Council Directive 93/42/EEC.

ATTENTION

This product is only to support your natural motion. It is not allowed to place your total body weight. Clean and dry the area where the holding grip is to be mounted. Read the operating instructions, that you know the limitations of the holding grip, before using it. Test the holding grip before using it.

Liability Warning

Before using, know your own limitations and the limitations of the wall and the mobile holding grip. Activities involving motion or height create the possibility of accidental injury. You the user assumes all risk of injury due to use. The mobile holding grip is sold under this condition.

The Mobile Grab Rail must be assembled in such a way, that the warning print can be read during use.

Designated use

The Mobile Vacuum Suction Grab Rail has been designed as a rail for wide independent use in conjunction with showers, baths, toilets and washbasins. It is intended to assist the body's natural movement, e.g. as an aid for standing up. It is not designed to support your whole body weight. The Mobile Grab Rail is intended to provide support for short periods only.

Scope of application

Due to the vacuum suction generated by the discs, the Mobile Grab Rail can only be attached to smooth and non-porous surfaces, which are pore- and steam tight, dry and free of oils and grease. On rough or porous surfaces, the vacuum hold diminishes or does not exist at all. Please bear in mind that tiles are not necessarily always smooth and steam-tight; test your tiles and the wall first (possibly with the help of another person).

As the vacuum suction discs are made from strong adhesive and flexible natural rubber, they can usually adapt to slightly textured tiles too.

The Mobile Vacuum Suction Grab Rail is ideal for short-term applications in households where there are no permanently fixed rails, e.g. after a stay in hospital.

The Mobile Suction Grab Rail can also be used where permanent rails cannot be mounted, e.g. in glass shower cubicles or in bathtubs. It can also be used when it is prohibited or impossible to drill holes in the wall (e.g. rented accommodation, water pipes or electrical cables behind the wall), when on visits to the doctor's surgery, public authorities, relatives or when travelling etc.

In addition, the Mobile Vacuum Suction Grab Rail can be used to determine the ideal position and height for mounting fixed rails, (i.e. before commencing any drilling and screwing).

For your information

In hotels and public establishments in Europe, tiles must be pore- and steam-tight (as per DIN/EN 159 & 176). According to DIN 18156, tile adhesives must have a minimum adhesive force of 0.5N/mm². For a minimum tile size of 130mm x 130mm x 0.5N/mm² multiplied by 2 tiles, this would result in an adhesive force of 16.900N, i.e. approx. 1.700kg. Unfortunately, one cannot always rely on this; therefore it is absolutely necessary to test the adhesive force of the respective tile(s) or other surface materials on the wall.

Furthermore, material aging, e.g. in prefabricated construction, can lead to a reduction in the adhesive power, therefore testing is essential (e.g. by abrupt force).

Suction power of the ROTH Mobile Holding Grips

Under optimal conditions, the Mobile Single Hand Grab Rail (Order No. 1403999) has a reliable suction power of about 687N (about 154lbs ; 70Kg).

Under optimal conditions, the extendable Mobile Grab Rails (Order Nos. 140052-1, 140052-2, 140052-3, 140052-6, 140052-8) have a reliable suction power of 585N (about 134lbs ; 60Kg).

Prerequisite: see under "Hints for Your Safety".

Hints for your safety

- Use the Mobile Vacuum Suction Grab Rail only if you are experienced in handling it or if someone who is experienced in handling it supervises you.
- Always check that the Mobile Grab Rail is functioning properly before each use. Pay particular attention to the rubber suction discs; they should be smooth and undamaged and should not have any fissures or stretch marks. **Never use mobile grab rails with damaged rubber discs or with limited suction power.**
- **Always check that the Mobile Grab Rail has a solid hold before use.**
- After lifting the vacuum levers, the vacuum suction produced can release completely or can partially remain, especially with unglazed tiles, natural stone, wood and/or ceramic coating that has been textured/grinded or given a rough grain to provide a non-slip finish.
- The Mobile Grab Rail should be used to support people for **short periods** only.
- If the Mobile Grab Rail is used as an aid for standing up, its suction power can release abruptly due to unfavourable circumstances (e.g. damaged or brittle rubber discs).
- Please bear in mind that the suction power of the rubber discs is dependent on the surface it is to be adhered to. Therefore please ensure that you check the stability of the surface (e.g. of the tile on the wall) before use.
- Attention! Use under water reduces the suction power.
- The force of the suction power stated earlier applies at normal room temperature (10-30°C). At very high temperatures, e.g. in a sauna, the suction power is reduced.
- The indicated force of the suction power also applies at about 300m above sea level. At higher altitudes, the suction power diminishes (by approx. 10% per further 1000m in height).
- Never expose the rubber discs to the sunlight for long periods otherwise they will quickly lose their elasticity. For this reason, never attach the grab rail to any surface (e.g. window) that will be illuminated from outside (UV radiation and heat generation).
- Do not place anything below the grab rail that could be damaged by the rail suddenly releasing and falling on it. Alternatively, remove the grab rail after use.
- **The Mobile Vacuum Suction Grab Rail should only be used with the tilting levers perfectly engaged.**
- Do not attach the Mobile Grab Rail to thin vehicle metal panels or to glass panes because they risk being deformed by the negative pressure created by the vacuum suction.

Attaching the Mobile Vacuum Suction Grab Rail

- Check the vacuum suction discs and the surface it is to be attached to.
- Clean the surface thoroughly (see Page 5, fig 1)
- Press the rubber discs firmly to the surface. The tilting vacuum levers must be loose at this point (fig. 2) and the whole of each rubber disc must lie on the surface.
- **Ensure that the rubber discs are completely on the tile area or other surfaces so that the discs do not cover any grouting lines or other joints. The discs should be positioned at least 10 mm away from all joints** (fig. 2, 10, 11). You can adjust the length of the rail (figs. 5, 8) in telescopic fashion. (Order Nos. 140052 – 1 – 3 and – 8).
- First press down one tilting lever and then the second (figs 2, 3, 6, 8). **The resistance of the negative pressure generated must be clearly felt.** The greater the resistance felt on the vacuum lever, the better the suction power of the Mobile Grab Rail. If you use the rail regularly, you will get a feel for the vacuum strength and the suction power generated.
- **Before each use, test the firm hold of the suction device (figs 4, 7). If the adhesion seems insufficient to you, release the rail and reattach again.** (Lifting and releasing the vacuum levers again can do this). If the required suction power has not been achieved, the rail should not be used under any circumstance.

Releasing:

- Hold the tilting lever between finger and thumb and lift, using thumb to prevent recoil.
- The rubber discs will return to their original flat position and the suction rail can be removed from the wall. If the rubber discs still remain attached, lift the discs on one side with a finger so that air can penetrate under them.
- Do not try to release the rubber discs with sharp-edged objects or tools; you will damage them.
- When not attached to the wall, store the Mobile Grab Rail with the rubber discs flat (i.e. with tilting levers disengaged).

Disassembly and assembly (only Order nos. 140052- 1-3,6,8)

- Disassembling – Press in the catch (see page 6, fig 12 – a (possibly with a sharp instrument) and pull the rail section from the suction housing.
- Assembly: Press in the catch on the rail (figs 13 – a) and push the rail into the suction housing until the metal catch clicks into the hole. Check that the catches have clicked safely into place by trying to pull the sections apart. If both catches have not gone into place, you should not use the rail.

Assembly: **Order nos. 140052-8 see page 5 fig. 8+9**

Maintenance:


- Always keep the rubber discs clear of oil and dust.
- Clean the rubber discs with standard household cleaning agents like washing up liquid or spirit. Basic cleaning with acetone (available in pharmacies) also increases the suction effect and is recommended at longer intervals (depending on the frequency and circumstances of use, e.g. the use of oily bath additives or heavy soiling). Never expose the rubber discs to sunlight for long periods otherwise they will quickly lose their elasticity.
- Check the rubber discs for possible deterioration in suction power due to material perish caused by aging, wear and tear, exposure to sunlight, not leaving discs in the flat position when not attached, stretching etc. If rubber discs are damaged or their suction power has deteriorated, replace them together with the complete suction housing. (Test on a window/glass pane or mirror). We recommend that each suction housing be checked thoroughly by an expert or replaced after about 2 years. (Spare parts nos. 140052-R or 140052-L, see page 6 fig. 14).

Waste disposal:

The product packaging should be recycled. The metal parts can be recycled as scrap metal. Waste disposal must be carried out according to the respective national statutory regulations. Please ask your local authority about local waste disposal companies.

Technical Data

| ROTH Order no. | Rail length A (with adjustable telescopic rail) in mm | Total length-B Length in mm | Grip Ø in mm | Weight in g |
|----------------|---|-----------------------------|--------------|-------------|
| 140052-1 | 210 – 335 | 435 – 560 | 35 | 775 |
| 140052-2 | 330 – 455 | 555 – 680 | 35 | 920 |
| 140052-3 | 430 – 555 | 655 – 780 | 35 | 1030 |
| 140052-6 | 340 | 565 | 35 | 900 |
| 140052-8 | 205 - 225 | 430–450 | 26 | 650 |
| 1403999 | ca. 100 | 325 | 33 | 550 |

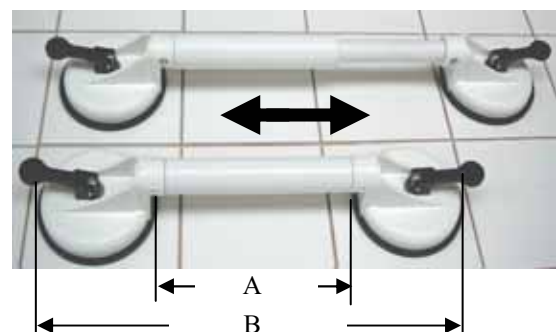inches measurements see page 8

Subject to technical modifications

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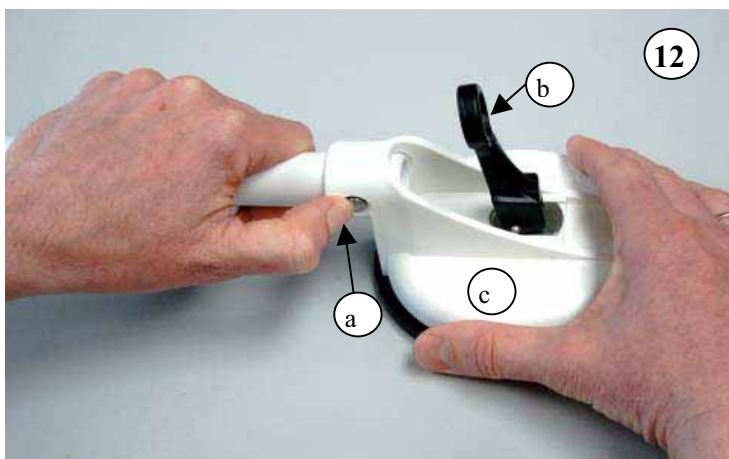
ATTENTION

- This product is only to support your natural motion. It is not allowed to place your total body weight. Clean and dry the area where the holding grip is to be mounted. Read the operating instructions, that you know the limitations of the holding grip, before using it. Test the holding grip before using it.

Liability Warning

- Before using, know your own limitations and the limitations of the wall and the mobile holding grip. Activities involving motion or height create the possibility of accidental injury. You the user assumes all risk of injury due to use. The mobile holding grip is sold under this condition.
- The Mobile Grab Rail must be assembled in such a way, that the warning print can be read during use.

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**ROTH GmbH D-72213 Altensteig, Kohlbergstraße 5 ,